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### **Références**

- AAMI TIR12, Designing, testing, and labeling reusable medical devices for re-processing in health care facilities: A guide for medical device manufacturers.

- ANSI/AAMI ST79, Comprehensive guide to steam sterilization and sterility assurance in health care facilities.
- ISO 17665, Sterilization of health care products – Moist heat.
- ISO 15883 serie, Washer-disinfectors
- ISO 17664, Sterilization of medical devices – Information to be provided by the manufacturer for the processing of resterilizable medical devices.

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---

## EN - Instructions for use, cleaning, and sterilisation of SPINE INNOVATIONS instruments.

Do not hesitate to consult your supplier's sales department if you need further information. This document is not exhaustive, nor is it a technical surgical manual containing detailed implantation information. Always refer to the documents provided.

### General information

- This leaflet is applicable to all reusable medical devices manufactured and distributed by SPINE INNOVATIONS and/or its distributors and must be read carefully.
- Both new and old instruments must be processed with care in accordance with these instructions before use.
- **These instructions are not applicable to SPINE INNOVATIONS medical devices supplied sterile, which must not be reused.**
- **Instrument cases do not provide a sterile barrier: sterilisation packaging must be used to keep the device sterile.**
- The instructions provided in this leaflet have been validated by SPINE INNOVATIONS. It is the health institution responsibility to ensure that reprocessing is carried out using suitable processing products and equipment and that the staff in charge of reprocessing has received proper training to obtain the expected result. The equipment and processes require regular validation and monitoring.
- Ancillary equipment is supplied non-sterile in transport containers. A transfer sheet (specifying the non-sterile status of the ancillary equipment) is supplied with each piece of ancillary equipment for incoming quality control and prior to sterilisation.
- SPINE INNOVATIONS equipment must be used exclusively for SPINE INNOVATIONS prostheses implantation and explantation.

### Intended purpose

Surgical Instruments are intended for the placement and positioning of the ESP implants and must be used in the manner described in the Surgical Technique related to the ESP implants.

The trial Implants are intended for the selection of the implant size best suited to the patient's anatomy. The instruments are intended for the implantation of the implants.

### Intended patient population

For the intended population of the instruments used during the surgical procedure refer to the associated implants instructions for use.

### Indications

For the indication of the instruments used during the surgical procedure refer to the associated implants instructions for use.

### Contraindications

For the contraindication of the instruments used during the surgical procedure refer to the associated implants instructions for use.

## Undesirable effects

For the undesirable effects and side effects during the surgical procedure refer to the implants instructions for use.

## Instructions for use

- A surgical procedure is available from the SPINE INNOVATIONS sales department or its distributors to provide surgeons with additional information on the recommended method for use. It is essential to read this document.
- It is recommended not to use this equipment directly after steam sterilization, and to wait until it has returned to room temperature. It is the responsibility of the operating room staff to check that SPINE INNOVATIONS instruments are compatible with the equipment available in the operating room prior to surgery.
- Ancillary equipment is designed to operate at room temperature.
- Some ancillary equipment consists of temporary invasive devices such as chisel or inclined impactor. These instruments must be used with caution. Their use can damage soft tissues or contact areas.
- This equipment, tested by SPINE INNOVATIONS, or its distributors, must be handled with care. Any damage to the equipment may give rise to potential malfunction.
- The equipment must only be used for its intended purpose in surgical procedures.

**SPINE INNOVATIONS recommends processing all devices in accordance with the combined manual decontamination/automatic cleaning instructions contained in this leaflet.**

## Warnings

- **The following precautions must be observed** by hospital staff working with contaminated or potentially contaminated medical devices. Pointed or sharp devices must be handled with the utmost caution.
- **Do not use metal brushes or scouring pads** during decontamination procedures. Their use could damage the instrument surface and finish. Use soft nylon bristle brushes and cleaning brushes of suitable sizes on devices to be processed.
- To facilitate the subsequent steps, **do not allow contaminated devices to dry before reprocessing.**
- Normal saline solution and cleaning/disinfection agents containing aldehyde, mercury, active chlorine, chloride, bromine, bromide, iodine or iodide are corrosive and **must not** be used.
- **Do not** use mineral oil or silicone lubricants, as they coat micro-organisms, prevent direct contact of the surface with the steam and are difficult to remove.
- **Automatic cleaning using only a washer/disinfector is not sufficient for spinal surgery instruments.** Thorough decontamination is required before an automatic cleaning procedure.

**N.B.: Integrity from sharp instrument i.e. rasp for discectomy and chisel must be inspected thoroughly after processing to ensure that the sharp edges are not damaged.**

SPINE INNOVATIONS recommends the use of an ISO 15883 compliant washer/disinfector for instrument cleaning and disinfection.

## Instructions

- Remove excess bodily fluids and tissues on the instruments with a disposable lint-free pad. Place the devices in a tray of distilled water or cover with a damp cloth.
- Instruments **must** be cleaned within 30 minutes following use to minimise the risk of drying before cleaning.
- Used instruments **must be** transported to the procurement department in sealed or covered containers to prevent further risk of contamination.

- Where applicable, instruments with multiple components must be dismantled for effective cleaning. Take care not to mislay any small screws and components.
- All cleaning agents must be prepared in accordance with the dilution and temperature recommended by the manufacturer. Softened tap water may be used to prepare cleaning agents. It is important to use the recommended temperatures for optimal cleaning agent performance.

**N.B.: Cleaning solutions must be prepared fresh.**

- The instruments must be removed from the metal or polymer trays when applying automatic cleaning procedures. Lids, cases and instrument trays must be cleaned separately.
- **The use of hard water must be avoided.** Softened tap water may be used for the initial rinse. The final rinses must be carried out with purified water to remove deposits on the instruments. One or more of the following processes may be used for water purification: ultrafiltration (UF), reverse osmosis (RO), de-ionisation or equivalent.
- Steam (moist heat) is the validated sterilisation method for SPINE INNOVATIONS instruments

### Processing Instructions

- Instruments can go through this cycle a number of times (read carefully the inspection section). If handled and used properly, Spine surgery instruments can be used for several years. However, this service life may be reduced rapidly due to poor handling or unsuitable protection. Any damaged instruments (for example with corrosion, excessive scratches, notches, residue, debris or unreadable laser markings) should be destroyed and replaced by a new instrument. In case of unexpected damage, please return the device to SPINE INNOVATIONS as a complaint..
- For users who do not use the recommended method, we recommend that they validate the methods they use by appropriate laboratory techniques.
- Instruments shall be decontaminated immediately after each use. They shall be thoroughly cleaned before sterilization and reintroduction into the sterile surgical field.
- Before being returned to SPINE INNOVATIONS, SPINE INNOVATIONS instruments are required to undergo all the decontamination, cleaning, disinfection, inspection and final sterilisation steps. The decontamination and sterilisation documentation must be provided with the instruments returned to SPINE INNOVATIONS.

### Cleaning

Automated cleaning process has been validated according to ISO 17664-1 and AAMI TIR 30. The automated cleaning cycle to be used is as follows:

1. Pre-cleaning - temperature < 45°C - duration 2min minimum
2. Cleaning - temperature ≈ 55°C - duration 5min minimum
3. Neutralizing - approximate duration 2min
4. Rinsing with cold tap water - approximate duration 2min
5. Thermal Disinfection with deionized water - temperature ≈ 90°C - duration 5min minimum
6. Dry in the cleaner – at least 22min
7. Perform a visual inspection on the instruments and verify that they are clean, dry, and in proper working order prior to sterilization.

The automated cleaner needs to be qualified according to the requirements defined in the ISO15883 standards. Cleaning solution (detergent) has to be selected by the hospital to be compatible with the use in automated cleaner/disinfector according to the ISO 15883 standard and on surgical instruments made out of metallic and/or plastic parts. Using an alkaline reagent is recommended, such as Neodisher or Mediclean Forte.

Doses, temperatures and durations may be adapted according to solution manufacturer instructions. Because of the use of some cleaning solutions, white traces may appear after drying. A discoloration has no adverse effect on titanium alloy devices.

### **Inspection before sterilisation**

- The cleaning procedures must be repeated until instruments are “thoroughly cleaned” and “visually clean”.
- Inspect each device carefully to ensure that any visible contamination has been removed. If contamination is observed, repeat the cleaning/disinfection procedure.
- Check that moving parts (e.g. hinges, connectors, sliding parts, etc.) can move to their full extent.
- Check that there is no instrument deformation
- When the instruments have been assembled, check that the devices are assembled correctly with the components in contact.
- Inspect devices for wear and tear: the device is not suitable for reuse if it looks corroded, discoloured, stained or damaged. Contact your sales representative if the device is defective.

### **Sterile packaging**

Commercially available medical grade steam sterilisation wrapping or pouches may be used to package individual instruments. The wrapping must be prepared according to the DIN EN ISO/ANSI AAMI ISO 11607-1/2 protective double wrapping technique. Trays and cases with lids may also be placed in an approved sterilisation container with a sterilisation sealing lid. Follow the sterilisation container manufacturer’s instructions for inserting and replacing sterilisation filters in sterilisation containers.

### **Instrument transport and storage cases and trays must be used under the following conditions:**

- If possible, all devices must be disassembled before they are placed in the case.
- All devices must be arranged to ensure the steam comes into contact with all the instrument surfaces. Instruments must not be stacked or placed in close contact (use the storage holders or silicone mats provided).
- The user must check that there has been no spillage of the instrument case contents once the devices have been arranged in the case.
- Only devices manufactured and/or distributed by SPINE INNOVATIONS may be placed in the SPINE INNOVATIONS instrument tray.

### **Sterilisation**

- Reusable surgical instruments must always be disinfected according to the specified procedure before sterilisation.
- The health institution is responsible for in-house instrument reassembly, inspection and packaging procedures following thorough cleaning so as to ensure sterilising steam penetration and suitable drying. Protection measures to be taken against sharp or potentially dangerous instrument edges must also be recommended by the health institution.
- Steam/moist heat sterilisation is the preferred method recommended for SPINE INNOVATIONS spine surgical instrument sets.
- The steriliser manufacturer’s recommendations must be followed **at all times**. When sterilising several instrument sets in a sterilisation cycle, take care not to exceed the maximum load specified by the manufacturer.
- The instrument sets must be prepared and packaged correctly in trays and/or cases so that the steam can penetrate and come into direct contact with all the surfaces.
- See the table below for the cycle sterilisation settings validated by SPINE INNOVATIONS to provide a sterility assurance level (SAL) of 10<sup>-6</sup>.

- Ethylene oxide or plasma sterilisation methods **must not** be used.

**Validated steam sterilisation settings:**

	<b>Cycle type</b>	<b>Temperature (°C)</b>	<b>Temperature (°F)</b>	<b>Exposure time</b>	<b>Drying time</b>
A	Prior vacuum	132°C	269.6°F	4 minutes	20 minutes
B	Prior vacuum	134°C	273.2°F	18 minutes	20 minutes
C	Prior vacuum	134°C	273.2°F	3 minutes	20 minutes

This procedure was validated by an independent laboratory in accordance with AAMI TIR12.

**Storage**

When handling packaged cases, take care to avoid damaging the sterile barrier. The health institution must define a storage shelf-life for packaged instruments based on the sterile packaging and the packaging manufacturer's recommendations.

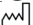
**IMPORTANT INFORMATION FOR HEALTH CARE ESTABLISHMENT PERSONNEL**

Any serious incident related to the device must be reported to the manufacturer and to the competent authority of the Member State of the user's residence.

**References**

- AAMI TIR12, Designing, testing, and labelling reusable medical devices for re-processing in health care facilities: A guide for medical device manufacturers.
- ANSI/AAMI ST79, Comprehensive guide to steam sterilisation and sterility assurance in health care facilities.
- ISO 17665, Sterilisation of health care products – Moist heat.
- ISO 15883 serie, Washer-disinfectors
- ISO 17664, Sterilisation of medical devices – Information to be provided by the manufacturer for the processing of resterilisable medical devices.

## Symbols meaning

	Manufacturer
	Consult instructions for use
	Keep dry
	Keep away from sunlight
	Do not use if the packaging is damaged and consult instructions for use
	Manufacturing date
<b>LOT</b>	Batch code
<b>REF</b>	Catalog number
<b>UDI</b>	Unique device identifier
<b>MD</b>	Medical device
	Non-sterile

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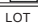
### Referenzen

- AAMI TIR12, Designing, testing, and labeling reusable medical devices for re-processing in health care facilities: A guide for medical device manufacturers.
- ANSI/AAMI ST79, Comprehensive guide to steam sterilization and sterility assurance in health care facilities.
- ISO 17665, Sterilization of health care products – Moist heat.
- ISO 15883 serie, Washer-disinfectors.
- ISO 17664, Sterilization of medical devices – Information to be provided by the manufacturer for the processing of resterilizable medical devices.

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### Referenties

- AAMI TIR12, Designing, testing, and labelling reusable medical devices for re-processing in health care facilities: A guide for medical device manufacturers.
- ANSI/AAMI ST79, Comprehensive guide to steam sterilisation and sterility assurance in health care facilities.

- ISO 17665, Sterilisation of health care products – Moist heat.
- ISO 15883 serie, Washer-disinfectors.
- ISO 17664, Sterilisation of medical devices – Information to be provided by the manufacturer for the processing of resterilisable medical devices.

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- AAMI TIR12, Designing, testing, and labelling reusable medical devices for re-processing in health care facilities: A guide for medical device manufacturers.
- ANSI/AAMI ST79, Comprehensive guide to steam sterilisation and sterility assurance in health care facilities.
- ISO 17665, Sterilisation of health care products – Moist heat.
- ISO 15883 serie, Washer-disinfectors.
- ISO 17664, Sterilisation of medical devices – Information to be provided by the manufacturer for the processing of resterilisable medical devices.

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#### Reference

- AAMI TIR12, Designing, testing, and labelling reusable medical devices for re-processing in health care facilities: A guide for medical device manufacturers.
- ANSI/AAMI ST79, Comprehensive guide to steam sterilisation and sterility assurance in health care facilities.
- ISO 17665, Sterilisation of health care products – Moist heat.
- ISO 15883 serie, Washer-disinfectors.
- ISO 17664, Sterilization of medical devices – Information to be provided by the manufacturer for the processing of resterilizable medical devices.

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- AAMI TIR12, Designing, testing, and labeling reusable medical devices for re-processing in health care facilities: A guide for medical device manufacturers.
- ANSI/AAMI ST79, Comprehensive guide to steam sterilisation and sterility assurance in health care facilities.
- ISO 17665, Sterilisation of health care products – Moist heat.
- ISO 15883 serie, Washer-disinfectors.
- ISO 17664, Sterilisation of medical devices – Information to be provided by the manufacturer for the processing of resterilisable medical devices.

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- AAMI TIR12, Designing, testing, and labeling reusable medical devices for re-processing in health care facilities: A guide for medical device manufacturers.
- ANSI/AAMI ST79, Comprehensive guide to steam sterilization and sterility assurance in health care facilities.
- ISO 17665, Sterilizarea de health care products – Moist heat.
- ISO 15883 serie, Washer-disinfectors.
- ISO 17664, Sterilization of medical devices – Information to be provided by the manufacturer for the processing of resterilizabile medical devices.

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#### Referenser

- AAMI TIR12, Designing, testing, and labelling reusable medical devices for re-processing in health care facilities: A guide for medical device manufacturers.
- ANSI/AAMI ST79, Comprehensive guide to steam sterilisation and sterility assurance in health care facilities.
- ISO 17665, Sterilisation of health care products – Moist heat.
- ISO 15883 serie, Washer-disinfectors.
- ISO 17664, Sterilisation of medical devices – Information to be provided by the manufacturer for the processing of re-sterilisable medical devices.

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## Referensi

- AAMI TIR12, Designing, testing, and labelling reusable medical devices for re-processing in health care facilities: A guide for medical device manufacturers.
- ANSI/AAMI ST79, Comprehensive guide to steam sterilisation and sterility assurance in health care facilities.
- ISO 17665, Sterilisation of health care products – Moist heat.
- ISO 15883 series, Washer-disinfectors
- ISO 17664, Sterilisation of medical devices – Information to be provided by the manufacturer for the processing of resterilisable medical devices.

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- AAMI TIR12, Designing, testing, and labelling reusable medical devices for re-processing in health care facilities: A guide for medical device manufacturers. (AAMI TIR12, 《可重复使用医疗器械在医疗保健机构中进行再处理的设计、测试和标识: 医疗器械制造商指南》。)
- ANSI/AAMI ST79, Comprehensive guide to steam sterilisation and sterility assurance in health care facilities. (ANSI/AAMI ST79, 《医疗保健机构中蒸汽灭菌和无菌保证综合指南》。)
- ISO 17665, Sterilisation of health care products – Moist heat. (ISO 17665, 《医疗保健产品灭菌 – 湿热》。)
- ISO 15883 serie, Washer-disinfectors (ISO 15883 系列, 《清洗消毒器》)
- ISO 17664, Sterilisation of medical devices – Information to be provided by the manufacturer for the processing of resterilisable medical devices. (ISO 17664, 《医疗器械灭菌 – 制造商为可重复灭菌医疗器械的处理需提供的信息》。)

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