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


Spineway IJU-TP-Revo - 12/2020

Cancel and replace No-TP-8

**twin peaks**

Instruction for use

Latest revision: December 2020

First EC marking date: 2006

Manufacturer: SPINeway

CAUTION: USA Federal law restricts this device to sale by or on the order of a physician.

1 DEVICE DESCRIPTION AND COMPOSITION

The twin peaks lumbar interbody fusion system is an internal spinal fixation system used to provide stabilization of the spine while facilitating fusion.

The twin peaks cages are inserted by impaction in the disc space by posterior approach.

The twin peaks cages are used with dedicated instruments which allow for improved technique and removal. Use surgical technique manual to support you in the choice of the instruments.

The twin peaks cages are supplied for single use.

Implant material: Polyetheretherketone (PEEK Optima LT1, ASTM F-2026) and Tantalum (ASTM F-2026).

2 INDICATIONS FOR USE

For the treatment of lumbar interbody fusion system is indicated for the treatment of degenerative disc disease.

The twin peaks lumbar interbody fusion system is used to stabilize the spine while facilitating fusion.

The twin peaks cages are used with dedicated instruments which allow for improved technique and removal. Use surgical technique manual to support you in the choice of the instruments.

The twin peaks cages are supplied for single use.

3 STERILE

